

User Guide

This guidance should be read in conjunction with the IFU supplied with the specific Breathing Circuit, the Flow Driver and Heater Humidifier




ALARM SETTINGS

Alarm	Function	Range
P Max	Sets high pressure alarm limit	0-25cm H ₂ O
P Min	Sets low pressure alarm limit	2-25cm H ₂ O
Apnoea	Triggers if no breath detected	20-60 Seconds
F Max	Sets maximum respiratory rate alarm	0-60 Breaths/min

THERAPY MENU


1. Patient respiratory rate and CPAP pressure display window
2. Patient respiratory wave form
3. Patient gas flow and oxygen settings button
4. Nebuliser function button
5. Alarm settings button
6. Therapy timer button

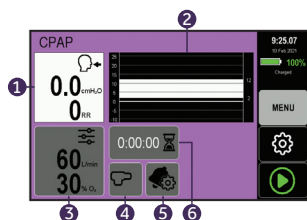


User Guide

This guidance should be read in conjunction with the IFU supplied with the specific Breathing Circuit, the Flow Driver and Heater Humidifier

Step 1

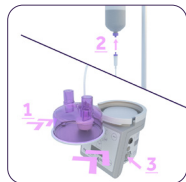

Step 2

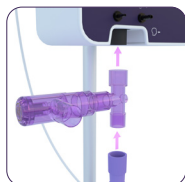

Step 3




Step 4




Step 5



Step 6

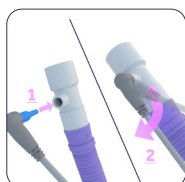

Step 7

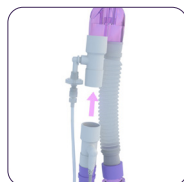


Step 8



1 PRESSURE TEST 1

Step 9

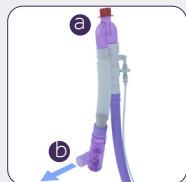

a. Start CPAP therapy & occlude patient end by your hand or with the red cap.


b. Verify gas flow through patient PEEP valve.

Step 10



Occlude patient PEEP valve.

Step 11

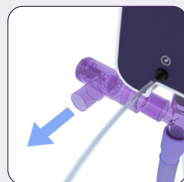

Verify gas flow through pressure relief valve.

Step 12

