

User Guide

This guidance should be read in conjunction with the IFU supplied with the specific Breathing Circuit, the Flow Driver and Infant Resuscitator.

TRANSITION TO HIGH FLOW OXYGEN THERAPY (HFOT)

Step 1

Disconnect the short T-piece adapter limb (a) from the heated inspiratory limb and remove the purple straight connector (b).



Step 2

Connect an appropriately sized HFOT nasal cannula.



Step 3

Fit the supplied overpressure relief valve between the humidification limb and the humidification chamber.



Step 4

Disconnect the humidification limb from the gas outlet on your resuscitation device.



Step 5

Connect the adapter to the humidification limb, if required.



Step 6

Connect the humidification limb to the auxiliary gas outlet or other appropriate gas supply source and activate the required gas flow rate setting.



Step 7

Secure the cannula to baby (a) and commence High Flow Oxygen Therapy (b).

