

User Guide

This guidance should be read in conjunction with the IFU supplied.

Step 1

- Size baby for NeoFlow® bonnet with sizing aid provided.
- Place to nape of baby's neck and bring forward to brow.



Step 2

Sizing down onto baby's brow to select the size and colour from the indicator.



Step 3

Select the bonnet size as per sizing.

AMBT100/XXS	Blue	Premature	up to 24cm
AMBT100/XS	White	Extra Small	24-28cm
AMBT100/S	Yellow	Small	28-31cm
AMBT100/M	Purple	Medium	31-34cm
AMBT100/L	Green	Large	34-38cm
AMBT100/XL	Red	Extra-Large	38-42cm

Step 4

Open bonnet and place to baby's nape. Ensure the soft side is against baby's head and coloured stitching is at the top.



Step 5

Fit to cover ears and wrap snugly over baby's brow.



Step 6

Secure with Velcro® tabs.

