


High Flow Oxygen Therapy – MaxVenturi®

Guidance for Use*

1. Select correct size of AquaNASE® nasal cannulae.
2. Apply AquaNASE® nasal cannulae and secure neck strap as shown overleaf.
3. Insert humidification chamber onto heater plate and spike water bag.
4. Attach circuit as shown a-d.
5. Connect heater cable and temperature probes.
6. Turn on heater and select invasive mode.
7. Set flow at 20L/min and select prescribed FiO₂ for 2 minutes.
8. Increase flow to max 70L/min as required monitoring SpO₂ and respiratory rate.

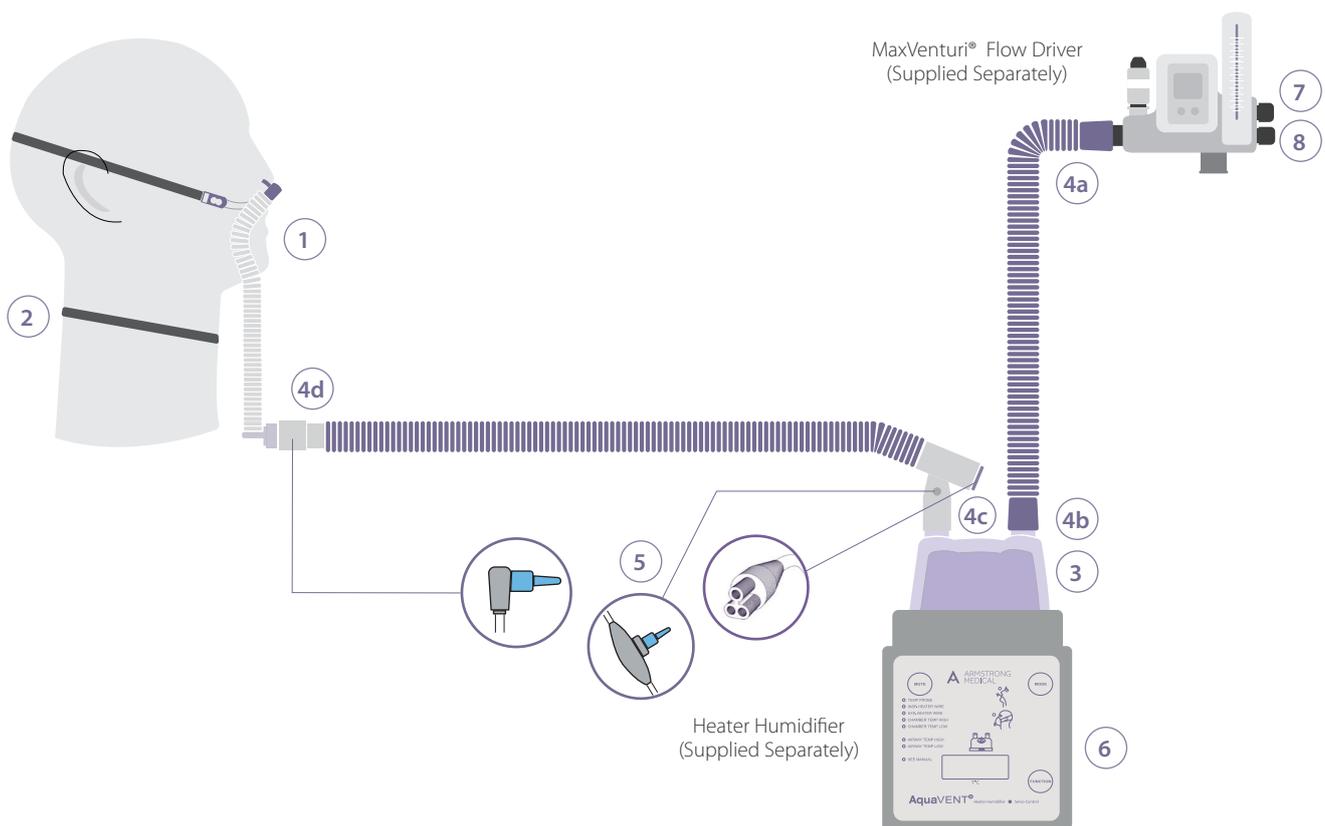

*This guidance should be read in conjunction with the IFU supplied with the specific Breathing Circuit, Heater Humidifier, AquaNASE® nasal cannulae and the Flow Driver.

AquaNASE[®]

Fitting Guide:

Step 1

Fit the looped neck support strap by placing over patients head and sliding down around the neck as shown.



Step 2

Adjust the looped neck support strap by pulling one strap through the clamp as shown.



Step 3

Place the elasticated strap around the patients head.



Step 4

Adjust the ends of the elasticated strap to secure the device.

