

## User Guide

This guidance should be read in conjunction with the IFU supplied.

### SET UP FOR NASAL INTERFACE

#### Step 1

Select the appropriate size of nasal cannula.

		NeoFlow Cannula
AMNC3001	● Premature	1-8 L/min
AMNC3002	● Neonatal	1-8 L/min
AMNC3003	● Infant	1-10 L/min
AMNC3004	● Large Infant	1-20 L/min
AMNC3005	● Paediatric	1-25 L/min



#### Step 2

(a) Pinch fixation pad to open the receiving channel.  
(b) Attach the fixation pad to the nasal cannula.



#### Step 3

Peel away the backing tape.



#### Step 4

Apply to baby in preferred position.

ⓘ Hold for 10 seconds.

