

V2 NIV Face Masks

FITTING GUIDE

SIZE COMPARISON

| | | | | | |
|-------------------------------|----|----|---|---|----|
| Eakin Healthcare V2 Mask Size | P | XS | S | M | L |
| Equivalent Competitor Size | XS | S | M | L | XL |

Most common sizes

MEASURING



① Always use the sizing gauge to determine the correct fit and maximise therapy performance.

STEP 1

Ensure the patient's facial muscles are relaxed and their jaw is closed.

Using the sizing gauge, place the top of the gauge at the **nasal bridge** and note the size marking where the bottom of the **chin bone** aligns with the gauge. This is your selected mask size.

APPLICATION



STEP 2

Remove the selected mask from its packaging. Attach the top two straps to the upper clips: the label should face outward.



STEP 3

Position the mask by rolling it from the chin upwards, covering the mouth and nose to create a seal.



STEP 4

Gently slide the headgear over the patient's head and clip in the bottom straps on both sides.



STEP 5

Facing the patient, adjust the Velcro straps to align the mask comfortably with facial contours.

LEAK CHECK



STEP 6

Ask the patient to exhale deeply and check for leaks around the nose, cheeks, or under the chin.



STEP 7

If required, adjust the strap at the crown of the head to correct minor leaks, this minimises discomfort and maintains a stable seal.



Non-Vented



Non-Vented + Anti-Asphyxia Valve



CO₂ Vented + Anti-Asphyxia Valve



Non-Vented Bronchoscopy

Scan to see our Hints & Tips

